


BRILLIANT Crios – How-to

Reinforced CAD/CAM composite for permanent restoration

STEP 1: TREATMENT OF BRILLIANT CRIOS RESTORATION

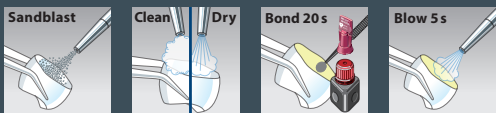

Note: Use ONE COAT 7 UNIVERSAL as bonding agent on BRILLIANT Crios.

STEP 2: BONDING AND LUTING TO SUBSTRATE

0 – 3 mm wall thickness

Bonding

- Light-curing
- Chemical-curing

Luting

- Light-curing, resin based
- Dual-curing, resin based
- Self-adhesive, dual-curing, resin based

3 – 5 mm wall thickness

Bonding

- Light-curing (with Activator where required)
- Chemical-curing

Luting

- Dual-curing, resin based
- Self-adhesive, dual-curing, resin based

STEP 3: FINAL CURING

Light-cure each surface for 30s to cure the adhesive on the BRILLIANT Crios restoration.



Note: This is no substitution for the instructions for use. More detailed information can be obtained on crios.coltene.com or via mail to info.ch@coltene.com

Do's

- Use of a dental dam is recommended.
- Follow the guideline for preparation and wall thickness.
- ONE COAT 7 UNIVERSAL adhesive must be applied to BRILLIANT Crios, whereas the use of other bonding agents on the tooth substance is possible.
- If a light-curing bonding is used on tooth surface, always light cure before placing the restoration.
- Only resin based luting materials are allowed to be used.

Don'ts

- Do not use silane.
- Do not use inorganic cements.
- Do not lute with light-curing opaque shades.
- Do not fire BRILLIANT Crios.
- Do not use hydrofluoric acid.

ONE COAT 7 UNIVERSAL

One-component light-cured universal adhesive

Developed to deliver excellent bonding performance, especially together with BRILLIANT Crios components. The material's self-conditioning ability created by MDP combined with nano filled technology improves mechanical properties and ensures long term stability of the final restoration.

- Only one universal bond for all indications and etching techniques
- Exceptional bonding values and excellent marginal seal
- Extremely economical: up to 300 applications per bottle



60019538 Intro Kit

- 1 × ONE COAT 7 UNIVERSAL, 5 ml
- 1 × Etchant Gél S, 2.5ml
- 1 × Accessories

60019540 Single Dose

- 1 × 50 Single Dose, 0.1 ml

60019539 Bottle Refill

- 1 × ONE COAT 7 UNIVERSAL, 5 ml

