

CanalPro™

Recommended irrigation protocol

Application of a rubber dam and disinfection of the tooth with **CanalPro NaOCl 3%**



Irrigation during preparation with **CanalPro NaOCl 3% or 6%**



Removal of smear layer after preparation: 1 min. irrigation with **CanalPro EDTA 17%**



CanalPro NaOCl 3% or 6% (at least 10 ml 5 min.)

Canal Pro CHX 2% (at least 10 ml and 5 min.)

CanalPro NaOCl 3% or 6% (at least 10 ml 5 min.)



Neutralisation of **NaOCl** with 0.9% NaCl solution/alcohol



Irrigation with **CanalPro CHX 2%**
(at least 10 ml and 5 min.)



Drying with **ROEKO / HyFlex paper points** and/or **ROEKO Surgitip-endo**



Obturation with **ROEKO GuttaFlow2 / bioseal**

Important! After the use of **NaOCl** and before the use of **CHX**, the root canal must be neutralised via intermediate irrigation (e.g. with NaCl or alcohol).

Please note: Irrigation with NaCl is usually recommended before measuring the length with apex locators.

CanalPro™ SyringeFill order information



CanalPro™ SyringeFill	REF
CanalPro™ SyringeFill Station (1 × bottle holder incl. base, 3 × CanalPro SyringeFill Cap)	60031347
CanalPro™ SyringeFill Holder (1 × bottle holder for wall mounting, 3 × CanalPro SyringeFill Cap)	60031348
CanalPro™ SyringeFill Cap (cap with integrated Luer lock)	60019662
CanalPro™ Irrigation solutions	
CanalPro™ NaOCl 3%, 500 ml	60019655V
CanalPro™ NaOCl 6%, 500 ml	60019656V
CanalPro™ EDTA 17%, 500 ml	60019651
CanalPro™ EDTA 17%, 100 ml	60019654
CanalPro™ CHX 2%, 500 ml	60019658
CanalPro™ CHX 2%, 100 ml	60019659
CanalPro™ colour syringes	
CanalPro™ colour 5ml syringes red, 50 pcs.	60019321
CanalPro™ colour 5ml syringes blue, 50 pcs.	60019322
CanalPro™ colour 5ml syringes yellow, 50 pcs.	60019323
CanalPro™ colour 5ml syringes white, 50 pcs.	60019324
CanalPro™ colour 10ml syringes red, 50 pcs.	60011173
CanalPro™ colour 10ml syringes blue, 50 pcs.	60011174
CanalPro™ colour 10ml syringes yellow, 50 pcs.	60011175
CanalPro™ colour 10ml syringes white, 50 pcs.	60011176
Irrigation tips	
CanalPro™ Slotted-End Tips 27 ga, 100 pcs.	60011167
CanalPro™ Slotted-End Tips 30 ga, 100 pcs.	60011168
CanalPro™ Side-Port Tips 27 ga, 100 pcs.	60011169
CanalPro™ Side-Port Tips 30 ga, 100 pcs.	60011170

Coltène/Whaledent AG
 Feldwiesenstrasse 20
 9450 Altstätten / Switzerland
 T +41 71 757 5300
 info.ch@coltene.com

Coltène/Whaledent GmbH + Co. KG
 Raiffeisenstraße 30
 89129 Langenau / Germany
 T +49 7345 805 0
 info.de@coltene.com

Coltène/Whaledent Ltd.
 The President Suite
 Kendal House, Victoria Way
 Burgess Hill, West Sussex
 RH15 9NF/GB
 T +44 1444 235486
 info.uk@coltene.com

 **COLTENE**

www.coltene.com