

Temporary Cement


TempoSIL[®] 2

ONE temporary PVS cement material for different surfaces and indications

- Secure adhesion of provisional restorations to natural tooth structure, core build up material and metal alloys as well as zirconia
- Simple application as well as easy removal
- Fast setting time
- Two shades for esthetic applications

TempoSIL 2 guarantees optimal adhesion on different surfaces as well as superior marginal integrity. The ideally balanced elasticity of the TempoSIL 2 formulation ensures for trouble-free removal of excess temporary cement. For esthetic applications, TempoSIL 2 is now available in 2 different shades: *white* and *dentin*.

Clinical case: provisional restoration

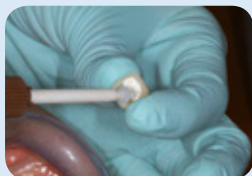

Apply TempoSIL 2 directly into the restoration using the Oral Tip.



Seat the restoration using gentle pressure.



Hold the restoration in place between 30-60 s.



After setting, remove excess TempoSIL 2 cement.




Cemented provisional restoration.

Clinical case: implant abutments with bridgework



Positioned implant abutments

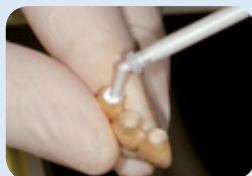

Apply TempoSIL 2 with the Oral Tip directly into the restoration.



Seat the zirconia restoration by applying gently pressure.




After setting, remove excess TempoSIL 2 material.



Cemented zirconia restoration.

Superior marginal integrity


Evaluation of dye penetration into provisional crowns cemented with TempoSIL 2 after 2 weeks at 37 C warm water followed by 24-hour dye penetration.*

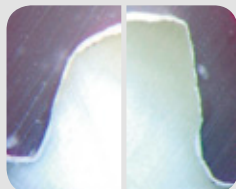

The superior marginal seal of TempoSIL 2 prevents dye penetration and minimizes wash outs.


Secure adhesion

The innovative formulation of TempoSIL 2 delivers micro-mechanical retention on different surfaces.*

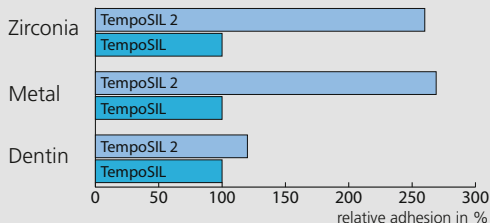

*Internal measurement

Trouble-free removal

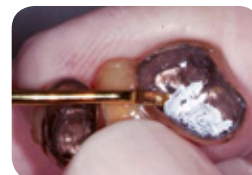

Removable without leaving any residue.

Order Information

3686 TempoSIL 2 Trial Pack
1 x TempoSIL 2 dentin 11 g; 1 x TempoSIL 2 white 11 g
15 x Mixing Tips Brown long fine

6727 TempoSIL 2 dentin Intro Kit
2 x TempoSIL 2 dentin 11 g; 15 x Mixing Tips Brown long fine

6729 TempoSIL 2 white Intro Kit
2 x TempoSIL 2 white 11 g; 15 x Mixing Tips Brown long fine

6728 TempoSIL 2 dentin Refill
4 x TempoSIL 2 dentin 11 g

6730 TempoSIL 2 white Refill
4 x TempoSIL 2 white 11 g

6749 Mixing Tips/Oral Tips
40 x Mixing Tip Brown long fine

© 2015 Coltène/Whaledent AG • www.coltene.com

Coltène/Whaledent AG
Feldwiesenstrasse 20
9450 Altstätten / Switzerland
Tel +41 71 757 5300
Fax +41 71 757 5301
info.ch@coltene.com

Coltène/Whaledent Inc.
235 Ascot Parkway
Cuyahoga Falls, Ohio 44223 / USA
Tel +1 330 916 8800
Fax +1 330 916 7077
info.us@coltene.com

Coltène/Whaledent Ltd.
The President Suite-A
Kendal House, Burgess Hill
Victoria Way
West Sussex, RH15 9NF/U.K.
Tel. +44 (0)1444 235486
Fax +44 (0)1444 870640
info.uk@coltene.com

COLTENE