

# STEP BY STEP PLUNGER EXCHANGE


from System 50 ml to 75 ml

**1**



To remove the screws on existing 50 ml plunger a Phillips head screwdriver is needed.

**2**

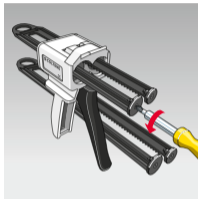

Use the Phillips head screwdriver to remove the screws by turning them to the left.

**3**



Once the screws have been removed, pull off the round plastic stoppers from the end of the plunger.



# STEP BY STEP PLUNGER EXCHANGE

from System 50 ml to 75 ml

**4**




Press the lever and remove the 50 ml plunger by sliding it through the impression gun.

**5**



Pick up the 75 ml plunger, remove the screws and round plastic stoppers from the plunger before inserting it in the impression gun.

**6**

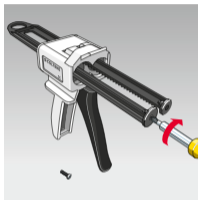

Place the round stoppers into the end of the 75 ml plunger rotating them, if necessary. Use the Phillips head screwdriver to insert the two screws. The 75 ml plunger installation is complete.