

GI-MASK Universal Separator

Instructions for use

EN

Definition

GI-MASK Universal Separator is a separating liquid for addition- and condensation type silicones in dental laboratories.

Type of material

Solution of fluorocarbon resins in isohexane.

Indication

GI-MASK Universal Separator is applied for easy removal of the gingival mask from the matrix impression or the silicone matrix material.

Important

Always wear gloves.

Contra-indications

Provided that the product is used as directed, there are no known contra-indications.

Application

1. Spray whole surface of the matrix impression or the silicone matrix material evenly with GI-MASK Universal Separator. Care about good ventilation
2. Air dry for at least 5 minutes.
3. Use the GI-MASK material as directed
4. As soon as the GI-MASK material has set, remove carefully matrix impression or the silicone matrix material.

Shelf life and storage

Recommended storage conditions of GI-MASK Universal Separator is 15-23 °C/59-73 °F and 50% relative air humidity in well sealed container. GI-MASK Universal Separator fulfils its intended purpose at least until expiry date displayed on the package.

Marking

The expiry date and **LOT** number are shown on the package.

Precautions

Contains Hexane-mixture of isomers, isopropyl acetate, 2-Methylpentane (< 5% n-Hexane), propane, butane, isobutane, isopropyl acetate.



GHS02



GHS07



GHS09



GHS08

DANGER

H336	May cause drowsiness or dizziness.
H411	Toxic to aquatic life with long lasting effects.
H315	Causes skin irritation.
H304	May be fatal if swallowed and enters airways.
H222+H229	Extremely flammable aerosol. Pressurized container: may burst if heated.
EUH044	Risk of explosion if heated under confinement.
P210	Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
P211	Do not spray on an open flame or other ignition source.
P251	Do not pierce or burn, even after use.
P271	Use only outdoors or in a well-ventilated area.
P261	Avoid breathing mist/vapours/spray.
P273	Avoid release to the environment.
P280	Wear protective gloves and protective clothing.
P264	Wash all exposed external body areas thoroughly after handling.
P301+P310	IF SWALLOWED: Immediately call a POISON CENTER/doctor/physician/first aider.
P331	Do NOT induce vomiting.

P312	Call a POISON CENTER/doctor/physician/first aider/if you feel unwell.
P391	Collect spillage.
P302+P352	IF ON SKIN: Wash with plenty of water and soap.
P304+P340	IF INHALED: Remove person to fresh air and keep comfortable for breathing.
P332+P313	If skin irritation occurs: Get medical advice/attention.
P362+P364	Take off contaminated clothing and wash it before reuse.
P405	Store locked up.
P410+P412	Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F.
P403+P233	Store in a well-ventilated place. Keep container tightly closed.
P501	Dispose of contents/container to authorised hazardous or special waste collection point in accordance with any local regulation.

SAFETY DATA SHEET

www.coltene.com

Glossary



Consult instructions for use



Keep away from sun light



Temperature limitation



Expiry date

Coltène/Whaledent AG

Feldwiesenstrasse 20
9450 Altstätten/Switzerland
T +41 71 757 5300
F +41 71 757 5301
info.ch@coltene.com

COLTENE