


# Assembly Guide

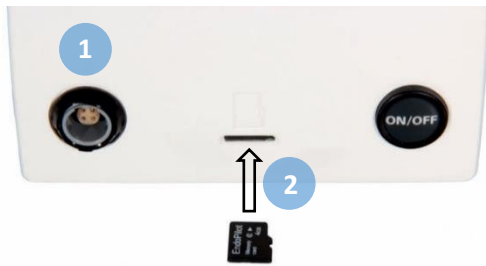

Connect the Basic Unit to the power supply (1) and check if the SD Card is placed in the SD Card slot (2) in the back of the basic unit



Connect the Contra-angle (Iso E-Connection) with the Motor (3) (Must **not** be rotated!).



Connect the motor with the display station (4).



Set the footpad on the floor within working distance of the basic unit.

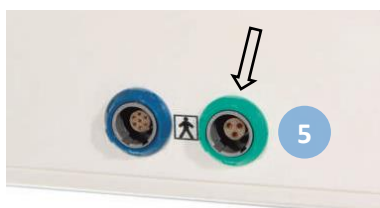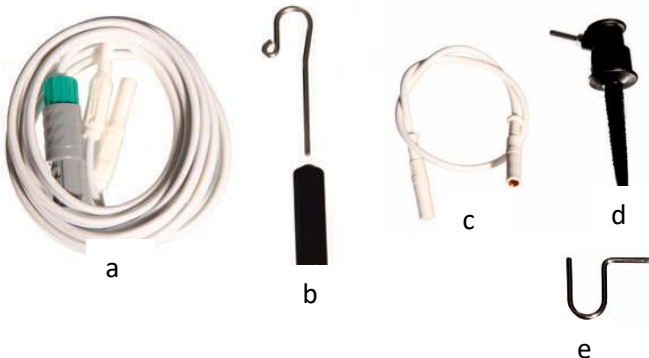

Assemble the Apex locator set and insert it into the Basic Unit (5).

Apex locator cable set consisting of:

- a – Measuring cable with plug
- b – Lip clip
- c – Cap for socket (for hygienic use of lip clip)
- d – Cable for file clamp (if applicable)
- e – File clamp (if applicable))



Switch on the basic unit (On/Off Button 6).

## First Steps



For setting different languages or changes in volume for all sounds except Apex sounds click here.




Apex



Preparation

For using only the Apex locator press the Apex Button.

For preparation, press the Preparation button.

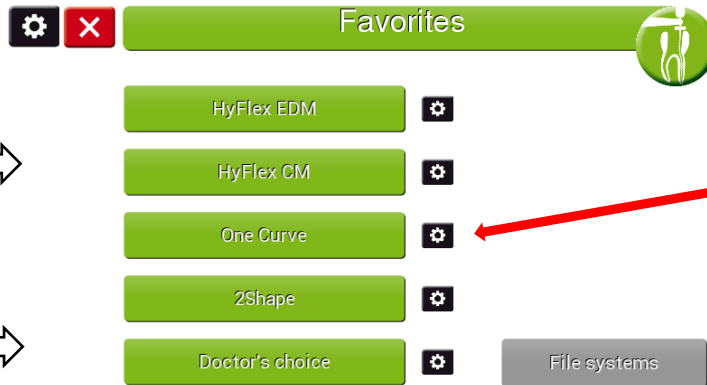

In the file system setup, you can choose the files to be displayed per sequence

After pressing the Preparation button, choose your desired file set or customize your own in the doctor's choice mode

For a first calibration of the motor click here. The motor should be calibrated after each change of the contra-angle.


For changes in the Apex settings (volume, sound, apex stop) also click here



Set the desired working length by touching the respective bar in the depicted canal.

Click here to turn off apex locator.

Set your individual settings for sounds, volume and apex stop here.

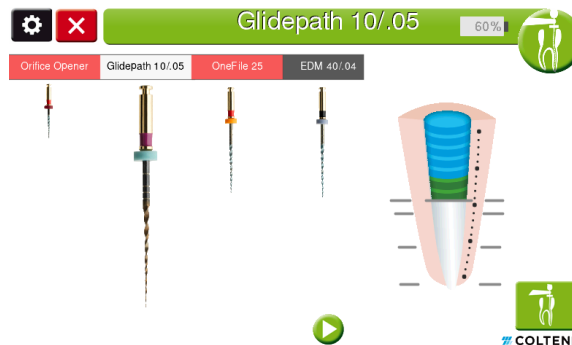

Click on this button for turning off the apex locator if wanted

Before starting the root canal treatment select your needed file, adjust the lip clip and you are ready to start the procedure.

Start the motor by pressing down the foot switch.




Apply gentle and continuous pressure to the file during preparation.

Hold the pressure until an acoustic signal sounds (long tone), which indicates that the canal must be irrigated.

Remove the file from the canal, irrigate the canal and then proceed further with preparation as described above.

When the acoustic signal for file change sounds (several short tones), please change the file.

# Doctor's choice – setup

In each file setup  in the Doctor's choice mode different movement types, torque and speed levels can be selected.

