



✗ Rewound (discard)



✓ After use



✓ After autoclaving



✓ New file

CROWN-DOWN TECHNIQUE - STEP BY STEP CARD

HyFlexTMCM

Coltène/Whaledent GmbH + Co. KG 

89129 Langenau/Germany

Coltène/Whaledent Inc.

Cuyahoga Falls, OH 44223/USA

www.coltene.com

Wiederaufbereitung/Reprocessing procedures/




www.hyflex.info


CROWN-DOWN TECHNIQUE
STEP BY STEP CARD

Examples for selection of file sequence depending
on the clinical situation

SMALL/MEDIUM
CANALS

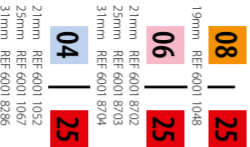

MEDIUM/LARGE
CANALS

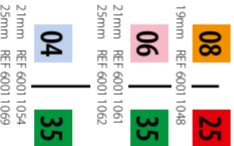

← Orifice Opener

Varying the taper
using a constant size

Take each file down
to resistance

04/25 GuttaPercha Point
REF 361 725

04/35 GuttaPercha Point
REF 361 735

Use a slow speed hand-
piece at 500 rpm.
Recommended torque
setting 2.4 N-cm / 24mNm