

STATIM® G4

Cassette Autoclaves



STERILIZATION AND DRYNESS

at speeds faster than conventional chambered autoclaves



Fast Cycle Times

STATIM 2000 G4 unwrapped cycle times are the world's fastest at 6 minutes. Wrapped cycles, including drying, can be completed in 35 minutes.

STATIM 5000 G4 can complete unwrapped cycles in as little as 9 minutes. Wrapped cycles, including drying, can be completed in 40 minutes.



Reliable

For over 25 years, STATIM's unmatched speed, reliability, and dependability have made it one of the world's best selling cassette autoclaves.



Capacity

STATIM G4 autoclaves not only offer fast sterilization times but also enough capacity to meet the needs of a busy practice.

STATIM 2000 G4 capacity allows for up to 5 pouches and the STATIM 5000 G4 allows for up to 12 pouches.



G4 Technology

SciCan's G4 equipment can connect and send information to your smart devices so you can have cycle data, instructions, and maintenance notifications at your fingertips.



Gentle

The fresh water feature guarantees fresh water with every cycle. State of the art steam technology is gentle on both solid and hollow instruments.



STATIM 2000 G4 Specifications

p/n G4-121101

Unit size (L x W x H):

19.5" x 16.25" x 6" / 49.5 cm x 41.5 cm x 15 cm

Cassette internal dimensions:

11" x 7" x 1.4" / 28 cm x 18 cm x 3.5 cm

Minimum shelf height for proper leveling:

8.5" / 22 cm

Weight without water:

46 lbs / 21 kg

Power consumption:

115 V, 60 Hz, 11A

	Unwrapped	Wrapped	Rubber / Plastic	Air Drying
Sterilization Time	3.5 minutes	10 minutes	15 minutes	Air drying only
Sterilization Temp	135° C	135° C	121° C	Air drying only
Total Cycle Time*	6 minutes	14 minutes	20 minutes	Air drying only

*Cycle time is dependent on instrument load and does not include drying.

STATIM 5000 G4 Specifications

p/n G4-201103

Unit size (L x W x H):

23.6" x 16.1" x 7.5" / 60 cm x 41 cm x 19 cm

Cassette internal dimensions:

15" x 7" x 3" / 38 cm x 18 cm x 7.5 cm

Minimum shelf height for proper leveling:

10" / 26 cm

Weight without water:



73 lbs / 33 kg

Power consumption:

115 V, 60 Hz, 11A

	Unwrapped	Wrapped	Rubber / Plastic	Heavy Duty Unwrapped
Sterilization Time	3.5 minutes	6 minutes	35 minutes	6 minutes
Sterilization Temp	132° C	132° C	121° C	132° C
Total Cycle Time*	9 minutes	17.5 minutes	45 minutes	12 minutes

*Cycle time is dependent on instrument load and does not include drying.

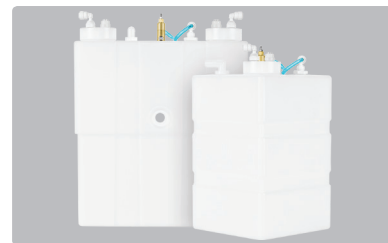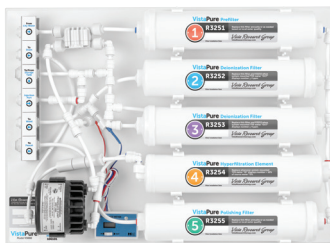

VistaPure™

Water Filtration

Provides two grades of high-quality water for autoclaves, dental bottles, ultrasonic cleaners and the rinse cycle of instrument washers.

V3000

VistaPure™ AutoFill

Replenish any new or older-model STATIM and BRAVO autoclave with distilled-quality water automatically, enabling truly hands-free convenience.

S3020 - AutoFill for STATIM

S3030 - AutoFill for BRAVO

VistaCool™

Direct-to-Drain System

Protects cabinets and drains—and saves valuable staff time—by cooling autoclave wastewater and sending it directly down the drain automatically.

V7501 - For One Autoclave

V7502 - For Two Autoclaves

Manufactured by:

SciCan Ltd.

1440 Don Mills Rd., Toronto, ON, Canada, M3B 3P9

Tel: 416-445-1600

Toll free: 1-800-667-7733

customerservice@scican.com

Distributed by:

SciCan Inc.

701 Technology Dr., Canonsburg, PA 15317, USA

Tel: 724-820-1600

Toll Free: 1-800-572-1211

uscustomerservice@scican.com

VistaPure™, VistaCool™ and DrainHub™ are manufactured by:

Vista Research Group

PO Box 321, Ashland, Ohio 44805-0321 USA

Tel: (419) 285-3927

VistaResearchGroup.com



ENSURE™

Monitoring for steam sterilization

An easy to use, in-office monitoring system for steam sterilization.

SCI-BISK-120 - Ensure BI Starter Kit

(25 BI's, Incubator & Log Book)



STATMAT™

Protect your STATIM

Securely seat cassettes atop your STATIM with a safe and hygienic silicone mat.

01-112867S - For STATIM 2000

01-112868S - For STATIM 5000