

roeko Gelatamp

Instructions for use

EN

Please read the Instructions for Use carefully before using the product. Keep for later reference.

1. PROPERTIES

Gelatamp (14 × 7 × 7 mm) is a haemostatically acting gelatine sponge to which 5% colloidal silver is added. Gelatamp is intended for the absorption of blood and stabilization of the coagulum to promote hemostasis, and the prevention of inflammatory complications after tooth extraction. In combination with colloidal silver, it avoids impaired wound healing and inhibits microbial colonization of the wound.

2. COMPOSITION

10 mg Gelatamp gelatine sponge contain:
 Hardened gelatine Ph. Eur. 9.5 mg
 Colloid silver Ph. Eur. 0.5 mg

3. INDICATION

Bleeding wound cavities after tooth extraction.

4. INSTRUCTIONS FOR USE

After opening the aluminium seal, Gelatamp is removed under aseptic conditions and is immediately ready for use. The size of the small sponge can be adjusted to fit the wound cavity if need be, without squeezing. Two Gelatamp plugs can be used for larger wounds.

The sponge is immediately placed dry and unsqueezed in the fresh, blood-filled wound cavity and should be allowed to absorb as much blood as possible (visual inspection). The sponge must not protrude over the inner gingival margin so as not to adversely affect the desired epithelisation. This constitutes definitive treatment of the alveolus and there is no need for subsequent rinsing.

5. STERILITY

Gelatamp is supplied gamma-sterilised. The sterility of undamaged, unopened packs is guaranteed until the first application. If packages are damaged prior to use they have to be discarded.

6. CONTRA-INDICATIONS

Infected, secretory and purulent wounds; hypersensitivity to silver or gelatine.

7. SIDE EFFECTS

Recovery may be delayed if used incorrectly, especially in the case of contaminated alveolar spaces.

8. INTERACTIONS WITH OTHER DRUGS

Concomitant application of alveolar pastes, wound plugs or other inserts and moistening with liquid substances or medication such as anion-active surfactants, antiseptics containing phenol, ethacridine lactate, dilute mineral acids and concentrated saline solutions can hinder wound recovery and should therefore be avoided. Please avoid contact with hydrogen peroxide as this will bleach the sponge.

9. STORAGE INSTRUCTIONS

Gelatamp possesses moisture-absorbing properties. The package must therefore be closed immediately after use and stored dry, protected from light. Keep medical devices out of the reach of children. Gelatamp has a shelf-life of 3 years. It can be used unopened up to the expiry date. Use within four weeks after opening. The product should no longer

be used once the date on the packaging and label has elapsed.

Unused sponges can easily be disposed of with laboratory waste whereas, the plastic container, lids and aluminium foil, etc. can be recycled.

10. AVAILABILITY

20 sponges REF 274002
 50 sponges REF 274007



Glossary

	Consult instructions for use
	Medical Device
	Legal Manufacturer
	Catalogue Number
	Manufacturing Date
	Expiry Date
	Batch Code
	Single use only
	Sterilized using irradiation
	Single sterile barrier system
	Do not use if package is damaged
	Keep dry
	Keep away from sun light
	Temperature limitation
	Bio material / Animal origin
	Medical substance
	Swiss representative
	Conformity mark Russia

Coltène/Whaledent
 GmbH + Co. KG
 Raiffeisenstraße 30
 89129 Langenau / Germany
 T +49 7345 805 0
 F +49 7345 805 201
 info.de@coltene.com

COLTENE