

Adhesive AC

Instructions for use

EN

Carefully read the instructions prior to use.

Product description

Coltène Adhesive AC is an autoclavable sprayable adhesive for the permanent bond between addition-polymerised silicones and plastic impression trays made of acrylates, polystyrene, polycarbonates.

Intended Use

A thin layer of COLTENE Adhesive AC serves as an adhesive between the impression material and the impression tray. It prevents the impression material from detaching from the impression tray during removal.

Composition

Main component: Polydimethylsiloxanecopolymer

Indications for Use

Used in combination with medical devices.

Contraindication

- In case of known allergies to the main ingredient of COLTENE Adhesive AC, do not use.
- COLTENE Adhesive AC is not compatible for use with metal trays or special plastic impression trays, in case of doubt test the acceptability.

Safety instructions

DANGER

- H225 Highly flammable liquid and vapour.
- H319 Causes serious eye irritation.
- H336 May cause drowsiness or dizziness.
- P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P337+P313 If eye irritation persists: Get medical advice / attention.
- EUH066 Repeated exposure may cause skin dryness or cracking.



GHS02 GHS07

WARNING

Contains ethyl acetate.

Use only in a well-ventilated areas!

Keep away from sources of ignitions!

Always wear gloves.

The use of COLTENE Adhesive AC may only be undertaken by specialist personnel such as dentists and dental assistants with special qualifications.

NOTICE

Because of the irreversible bond between additionpolymerized silicones and plastic impression trays, the plastic impression trays sprayed with COLTENE Adhesive AC after the impression is taken can no longer be used.

Side effects / Interactions

No harmful reactions or secondary effects on patients and / or dental personnel are known.

Users / patient groups

Application is carried out by dental professionals. Suitable for all patient groups.

PROPER USE

Spray (2 - 3 times) all plastic impression trays from a short distance with a thin, even layer of Coltène Adhesive AC, and let it dry for at least 1 min, or dry it using compressed air.

CLEANING & DISINFECTION

Cleaning of trays

Disposable plastic impression trays are not intended to be cleaned.

Shelf life and storage

COLTENE Adhesive AC fulfils the intended purpose at least until the expiry date, which is displayed on the containers, in a well-sealed bottle at a temperature of 15-23 °C / 59-73 °F and 50 % relative air humidity. Shelf life after first opening 3 months.

Disposal

Dispose of waste according to applicable legislation. Do not allow to enter into surface water or drains. Can be burnt together with household waste in compliance with official regulations in contact with approved waste disposal companies and with authorities in charge.

Reporting obligation

All serious incidents occurring in conjunction with this product must be reported immediately to the manufacturer as well as to the competent authority.

For SDS see
www.coltene.com



Glossary



Consult instructions for use



Keep away from sun light



Temperature limitation



Marking of Conformity Europe



Identification for Russia



Identification for Ukraine



Legal Manufacturer



Importer



Expiry Date



RX only



European Authorized Representative



Flame



Exclamation Mark



Batch Code



Medical Device



Reference Number



Unique Device Identifier

Coltène/Whaledent AG 
Feldwiesenstrasse 20
9450 Altstätten/Switzerland
T +41 71 757 53 00
F +41 71 757 53 01
info.ch@coltene.com