

# GI-MASK Universal Separator

## Instructions for use

EN

### Definition

GI-MASK Universal Separator is a separating liquid for addition- and condensation type silicones in dental laboratories.

### Type of material

Solution of fluorocarbon resins in isohexane.

### Indication

GI-MASK Universal Separator is applied for easy removal of the gingival mask from the matrix impression or the silicone matrix material.

### Important

Always wear gloves.

### Contra-indications

Provided that the product is used as directed, there are no known contra-indications.


### Application

1. Spray whole surface of the matrix impression or the silicone matrix material evenly with GI-MASK Universal Separator. Care about good ventilation
2. Air dry for at least 5 minutes.
3. Use the GI-MASK material as directed
4. As soon as the GI-MASK material has set, remove carefully matrix impression or the silicone matrix material.

### Shelf life and storage

Recommended storage conditions of GI-MASK Universal Separator is 15-23°C/59-73°F and 50% relative air humidity in well sealed container. GI-MASK Universal Separator fulfils its intended purpose at least until expiry date displayed on the package.

### Marking

The expiry date and  number are shown on the package.

### Precautions

Contains Hexane-mixture of isomers, isopropyl acetate



GHS02



GHS07



GHS09



GHS08

### DANGER

- H336 May cause drowsiness or dizziness.  
 H411 Toxic to aquatic life with long lasting effects.  
 H315 Causes skin irritation.  
 H304 May be fatal if swallowed and enters airways.  
 H222+H229 Extremely flammable aerosol. Pressurized container: may burst if heated.  
 EUH044 Risk of explosion if heated under confinement.  
 P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.  
 P211 Do not spray on an open flame or other ignition source.  
 P251 Do not pierce or burn, even after use.  
 P271 Use only outdoors or in a well-ventilated area.  
 P261 Avoid breathing mist/vapours/spray.  
 P273 Avoid release to the environment.  
 P280 Wear protective gloves and protective clothing.

- P264 Wash all exposed external body areas thoroughly after handling.  
 P301+P310 IF SWALLOWED: Immediately call a POISON CENTER/doctor/physician/first aider.  
 P331 Do NOT induce vomiting.  
 P312 Call a POISON CENTER/doctor/physician/first aider/if you feel unwell.  
 P391 Collect spillage.  
 P302+P352 IF ON SKIN: Wash with plenty of water and soap.  
 P304+P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing.  
 P332+P313 If skin irritation occurs: Get medical advice/attention.  
 P362+P364 Take off contaminated clothing and wash it before reuse.  
 P405 Store locked up.  
 P410+P412 Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F.  
 P403+P233 Store in a well-ventilated place. Keep container tightly closed.  
 P501 Dispose of contents/container to authorised hazardous or special waste collection point in accordance with any local regulation.

## SAFETY DATA SHEET

[www.coltene.com](http://www.coltene.com)

### Glossary



**Consult instructions for use**



**Keep away from sun light**



**Temperature limitation**



**Expiry date**

### Coltène/Whaledent AG

Feldwiesenstrasse 20  
 9450 Altstätten/Switzerland  
 T +41 71 757 5300  
 F +41 71 757 5301  
 info.ch@coltene.com

**COLTENE**