

**Bestellinformationen / Order information**

Größe / Size		REF
<b>Stay-put nicht imprägniert / non impregnated</b>		
x-fein / x-fine	<input type="checkbox"/>	521 000
fein / fine	<input type="checkbox"/>	521 001
mittel / medium	<input type="checkbox"/>	521 002
stark / thick	<input type="checkbox"/>	521 003

Länge / Length 183 cm

# roeko

## Stay-put

REF 5546

0

2

Größen / Sizes

**Coltène/Whaledent GmbH + Co. KG** Raiffeisenstraße 30  
89129 Langenau/Germany**Coltène/Whaledent Inc.**235 Ascot Parkway  
Cuyahoga Falls, OH 44223/USA[www.coltene.com](http://www.coltene.com)

3000 1394 04.16 xxx xx/xx

 **COLTENE**

# roeko

## Stay-put

Stay-put kombiniert die Qualität eines geflochtenen Retraktionsfadens mit der Formbarkeit eines hauchdünnen Drahtes. Die formbare Seele stabilisiert den Faden und bewirkt, daß er seine gewünschte Form behält. Stay-put läßt sich leicht in den Sulkus einbringen und bleibt in der gewünschten Position, wenn Sie Folgendes beachten:

Ein Stück Faden im Umfang des Zahnstumpfes abschneiden.  
Überlappen nicht notwendig.  
Den Faden **von der Mitte** aus in den Sulcus stopfen.  
Der Faden ist impägniert und nicht imprägniert erhältlich.



**TEST IT**

Stay-put combines the advantages of a braided retraction cord with the adaptability of a fine metal filament. The pliable core is so effective that the cord is not only easy to place in the sulcus but it stays there if you apply it as follows:

With sharp scissors cut a piece of cord corresponding to the circumference of the tooth. Overlapping is not necessary. Place the **centre** of the cord into the sulcus **first**. Next work out towards each end of the cord. The cord is available impregnated and non impregnated.