


SoloCem LUTING OF CROWNS, BRIDGES, INLAYS, ONLAYS

Optional: ONE COAT 7 UNIVERSAL on tooth

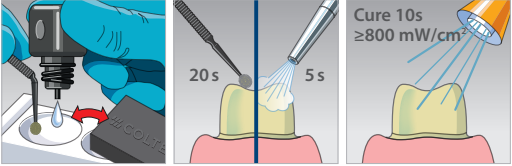


If improved bond strength is required

Composite, Metal, Zirconia


Silicate Ceramics

Preparation of restoration

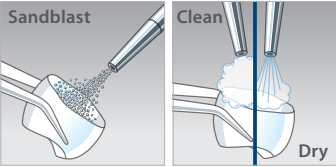


△ Manufacturer's IFU

Preparation of restoration

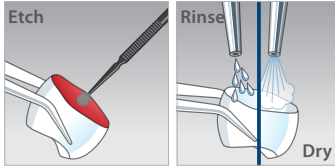


△ Manufacturer's IFU

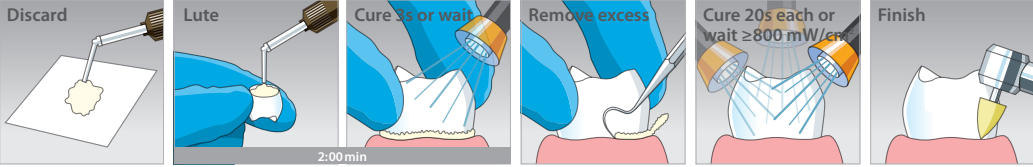

Discard Lute Cure 30s or wait Remove excess Cure 20s each or wait $\geq 800 \text{ mW/cm}^2$ Finish

2:00 min

SoloCem LUTING OF ROOT POSTS

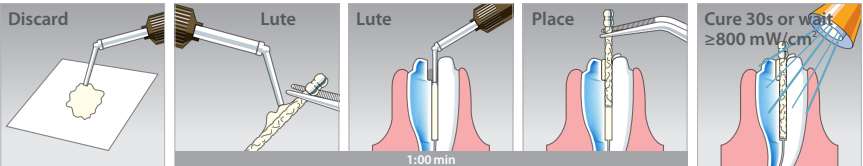

Discard Lute Lute Place Cure 30s or wait $\geq 800 \text{ mW/cm}^2$

1:00 min

△ For detailed instructions for preparation of root canal and post, please always consider the instructions for use provided by the post's manufacturer.

Do

- The use of a dental dam is recommended
- For improved bond strength, use ONE COAT 7 UNIVERSAL on the tooth
- Always consider the manufacturer's instructions for use of indirect restoration materials

Do Not

- Do not etch dentin with phosphoric acid
- Do not dry out dentin
- Do not use ONE COAT 7 UNIVERSAL without light curing

Questions?

Please contact your sales representative for any question or visit:

www.coltene.com

