

roeko Gelatamp Blister Pack

Instructions for use

EN

Please read the Instructions for Use carefully before using the product. Keep for later reference.

1. PROPERTIES

Gelatamp (14 × 7 × 7 mm) is a haemostatically acting gelatine sponge to which 5 % colloidal silver is added. Gelatamp is intended for the absorption of blood and stabilization of the coagulum to promote hemostasis, and the prevention of inflammatory complications after tooth extraction. In combination with colloidal silver, it avoids impaired wound healing and inhibits microbial colonization of the wound.

2. COMPOSITION

10 mg Gelatamp gelatine sponge contain:
 Hardened gelatine Ph. Eur. 9.5 mg
 Colloid silver Ph. Eur. 0.5 mg

3. INDICATION

Bleeding wound cavities after tooth extraction.

4. INSTRUCTIONS FOR USE

After opening a blister, Gelatamp is removed under aseptic conditions and is immediately ready for use. The size of the small sponge can be adjusted to fit the wound cavity if need be, without squeezing. Two Gelatamp plugs can be used for larger wounds. The sponge is immediately placed dry and unsqueezed in the fresh, blood-filled wound cavity and should be allowed to absorb as much blood as possible (visual inspection). The sponge must not protrude over the inner gingival margin so as not to adversely affect the desired epithelisation. This constitutes definitive treatment of the alveolus and there is no need for subsequent rinsing.

5. STERILITY

Gelatamp is supplied gamma-sterilised. The sterility of undamaged, unopened packs is guaranteed. If packages are damaged prior to use they have to be discarded.

6. CONTRA-INDICATIONS

Infected, secretory and purulent wounds; hypersensitivity to silver or gelatine.

7. SIDE EFFECTS

Recovery may be delayed if used incorrectly, especially in the case of contaminated alveolar spaces.

8. INTERACTIONS WITH OTHER DRUGS

Concomitant application of alveolar pastes, wound plugs or other inserts and moistening with liquid substances or medication such as anionactive surfactants, antiseptics containing phenol, ethacridine lactate, dilute mineral acids and concentrated saline solutions can hinder wound recovery and should therefore be avoided. Please avoid contact with hydrogen peroxide as this will bleach the sponge.

9. STORAGE INSTRUCTIONS

Gelatamp possesses moisture-absorbing properties and must therefore be stored under dry conditions. The package must be stored closed to ensure that it is protected from light. Keep medical devices out of the reach of children. The shelf life of Gelatamp is 3 years. The product should no longer be used after the date on the packaging and blister. Unused sponges can be disposed of easily with biological

waste and the blister packaging and cardboard box can be recycled.

10. AVAILABILITY

20 blisters

REF 274008



0483

Glossary



Consult instructions for use



Medical Device



Legal Manufacturer



Catalogue Number



Manufacturing Date



Expiry Date



Batch Code



Single use only



Sterilized using irradiation



Single sterile barrier system



Do not use if package is damaged



Keep dry



Keep away from sun light



Temperature limitation



Bio material / Animal origin



Medical substance



Swiss representative



Conformity mark Russia

Coltène/Whaledent
 GmbH + Co. KG
 Raiffeisenstraße 30
 89129 Langenau / Germany
 T +49 7345 805 0
 F +49 7345 805 201
 info.de@coltene.com