

BASIC OPERATION


1 Power unit ON.

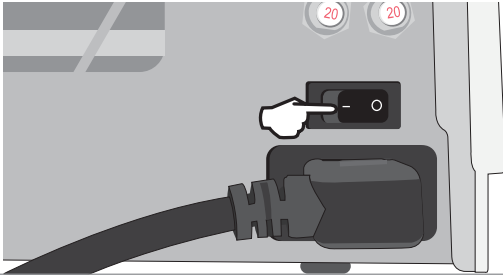
2 Fill with demineralised/distilled water.

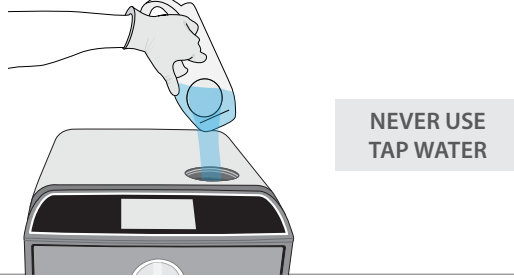
3 Load configurations.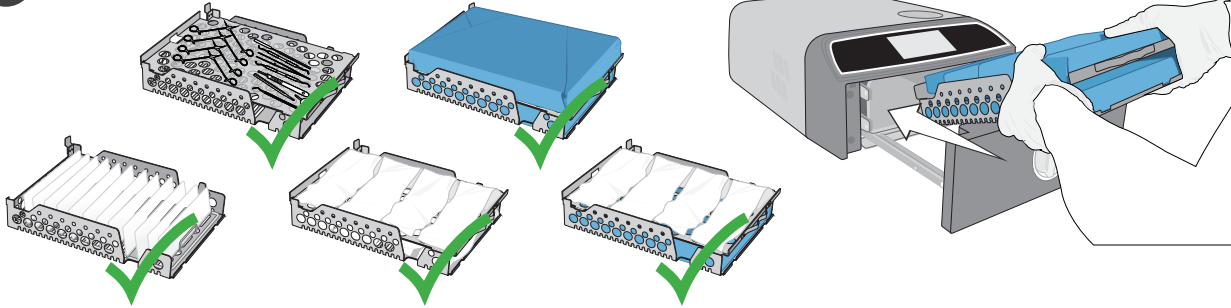


4 Close unit.

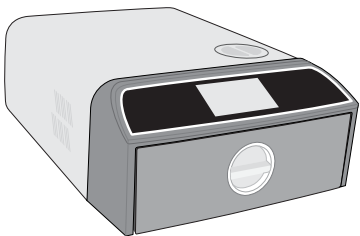
5 Run cycle.

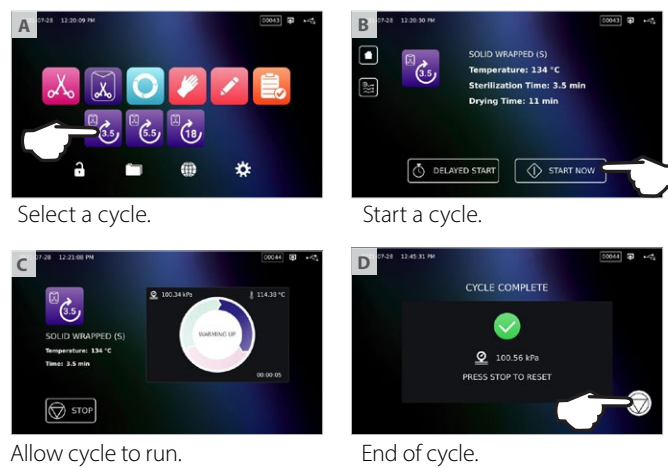
6 Open unit.

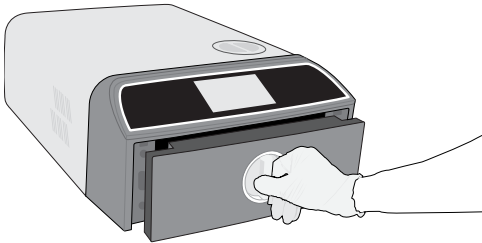7 Unload instruments.

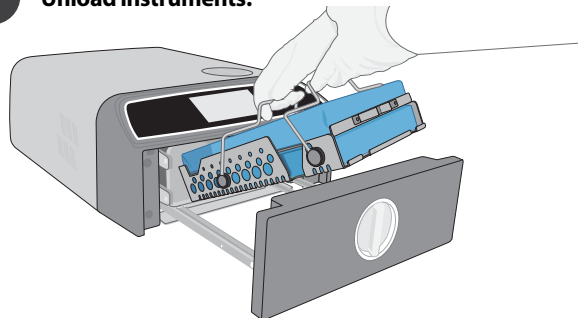
WARM-UP MODE

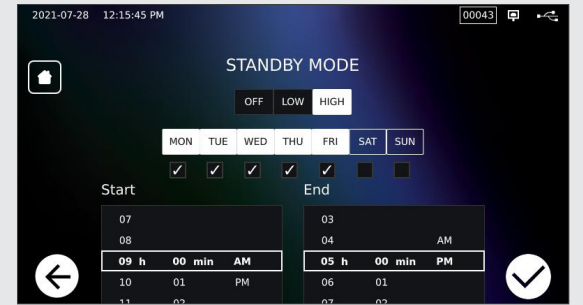
- STAND-BY LOW – For low to average use: 70°C / 158°F
- STAND-BY HIGH – For high use: 120°C / 248°F
- STAND-BY OFF (default) – For infrequent use. The wait time will be longer (up to 12 minutes from a cold start)


INSTRUCTIONAL VIDEOS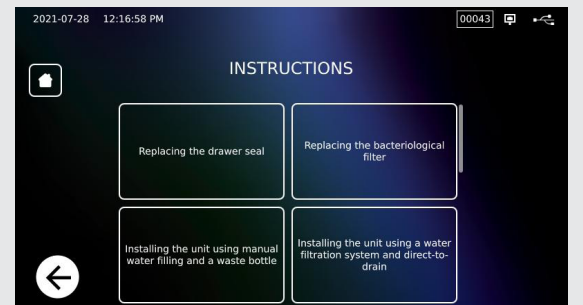


LED COLOUR

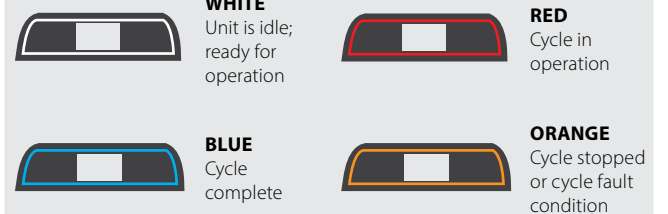LOCK ICON

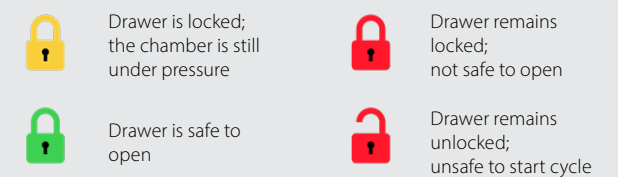TOUCHSCREEN

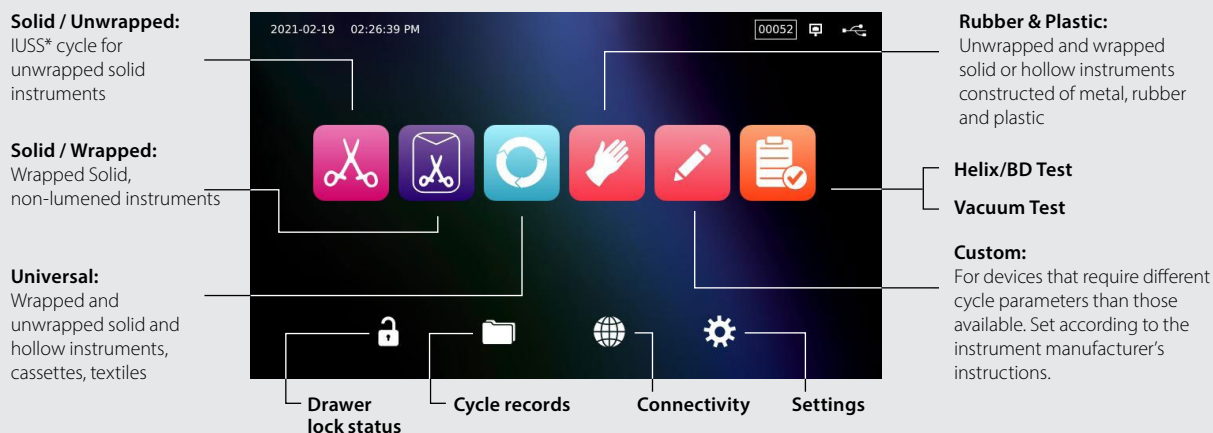*Immediate Use Steam Sterilization

MAINTENANCE SCHEDULE

Daily	Wipe the drawer seal . Clean external surfaces .	Monthly or 160 Cycles	Inspect the water reservoir filter (clean or replace). Clean the external fill water tank .
Weekly	Clean the chamber and trays . Drain water reservoir . Disinfect external surfaces . Inspect and clean the chamber filter (or replace if necessary).	Every 6 Months or 1,000 Cycles	Perform all monthly cleaning tasks . Replace bacteriological air filter . Replace drawer seal .
Every 2 Years or 2,000 Cycles	A complete maintenance of the autoclave including testing of the pressure relief valve and the power failure mode (by a SciCan-approved technician) is recommended.		

Spare Parts

01-1162925	Air Filter
01-1162935	Drawer Seal, 6L
01-1162945	Tray, 6L
01-1162965	Exhaust Tube
01-1162975	Extended Exhaust Tube
01-1162985	Direct-to-Drain
01-1162995	Water Filling
01-1163005	Waste Bottle
01-1163015	USB Memory Stick
01-1163025	Water Reservoir Cap
SCI-BDSK134V	Bowie-Dick Test Kit
01-1102825	Power Cord EU 16A/250V
01-1102855	Power Cord UK IND plug 16A/250V
01-116496	StatMat Silicone Mat STATIM B
01-1164975	Reservoir Chamber Filter
01-1164985	Pouch Rack Kit

IMPORTANT!

Users of this device must be sure to select the most appropriate sterilisation cycle according to the recommendations of the leading infection control authorities and local regulatory guidelines / recommendations.

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