

CanalPro Recommended irrigation protocol

Application of a rubber dam and disinfection of the tooth with **CanalPro NaOCl 3%**



Irrigation during preparation with **CanalPro NaOCl 3% or 6%**



Removal of smear layer after preparation: 1 min. irrigation with **CanalPro EDTA 17%**



CanalPro NaOCl 3% or 6% (at least 10 ml 5 min.)

Canal Pro CHX 2% (at least 10 ml and 5 min.)

CanalPro NaOCl 3% or 6% (at least 10 ml 5 min.)



Neutralisation of **NaOCl** with 0.9% NaCl solution/alcohol



Irrigation with **CanalPro CHX 2%**
(at least 10 ml and 5 min.)



Drying with **ROEKO / HyFlex paper points** and/or **ROEKO Surgitip-endo**



Obturation with **ROEKO GuttaFlow2 / bioseal**

Important! After the use of **NaOCl** and before the use of **CHX**, the root canal must be neutralised via intermediate irrigation (e.g. with NaCl or alcohol).
Please note: Irrigation with NaCl is usually recommended before measuring the length with apex locators.

CanalPro SyringeFill order information



CanalPro SyringeFill	REF
CanalPro SyringeFill Station (1 x bottle holder incl. base, 3 x CanalPro SyringeFill Cap)	60031347
CanalPro SyringeFill Holder (1 x bottle holder for wall mounting, 3 x CanalPro SyringeFill Cap)	60031348
CanalPro SyringeFill Cap (cap with integrated Luer lock)	60019662
CanalPro Irrigation solutions	
CanalPro NaOCl 3%, 500 ml	65002866
CanalPro NaOCl 6%, 500 ml	65002867
CanalPro EDTA 17%, 500 ml	60019651
CanalPro EDTA 17%, 100 ml	60019654
CanalPro CHX 2%, 500 ml	60019658
CanalPro CHX 2%, 100 ml	60019659
CanalPro colour syringes	
CanalPro colour 5ml syringes red, 50 pcs.	60019321
CanalPro colour 5ml syringes blue, 50 pcs.	60019322
CanalPro colour 5ml syringes yellow, 50 pcs.	60019323
CanalPro colour 5ml syringes white, 50 pcs.	60019324
CanalPro colour 10ml syringes red, 50 pcs.	60011173
CanalPro colour 10ml syringes blue, 50 pcs.	60011174
CanalPro colour 10ml syringes yellow, 50 pcs.	60011175
CanalPro colour 10ml syringes white, 50 pcs.	60011176
Irrigation tips	
CanalPro Slotted-End Tips 27 ga, 100 pcs.	60011167
CanalPro Slotted-End Tips 30 ga, 100 pcs.	60011168
CanalPro Side-Port Tips 27 ga, 100 pcs.	60011169
CanalPro Side-Port Tips 30 ga, 100 pcs.	60011170

Imprint / Publisher

Coltène/Whaledent AG
Feldwiesenstrasse 20
CH-9450 Altstätten
+41 71 757 5300
info.ch@coltene.com

Sales Contact

Coltène/Whaledent Ltd.
The President Suite
Kendal House, Victoria Way
Burgess Hill, West Sussex
RH15 9NF/UK

+44 1444 235486
info.uk@coltene.com

Service Center

service@coltene.com
00 800 11 0000 12 (UIFN)
Mon.-Fri. 8:00 to 17:00 MESZ