

# Adhesive

## Instruction for use

EN

**Carefully read the instructions prior to use.**

### Product description

COLTENE Adhesive is a universal adhesive for silicone-based impression materials for use in dentistry. It can be used for all conventional impression trays (e.g. metal).

### Intended Use

A thin layer of COLTENE Adhesive serves as an adhesive between the impression material and the impression tray. It prevents the impression material from detaching from the impression tray during removal.

### Composition

Main component: Polysiloxane

### Indications for Use

Used in combination with medical devices.

### Contraindication

In case of known allergies to the main ingredient of COLTENE Adhesive, do not use.

### Safety instructions

#### ⚠ DANGER

- H225 Highly flammable liquid and vapour.
- H400 Very toxic to aquatic life.
- H411 Toxic to aquatic life with long lasting effects
- P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P273 Avoid release to the environment.
- P370+P378 In case of fire: Use Water mist, carbon dioxide or alcohol-resistant foam to extinguish.
- P391 Collect spillage.
- P403+P235 Store in a well-ventilated place. Keep cool.
- P501 Dispose of contents/container in a safe place.



GHS02 GHS09

#### ⚠ WARNING

Contains Hexamethyldisiloxane

Use only in a well-ventilated areas!

Keep away from sources of ignitions!

Always wear gloves.

Close bottle immediately after use!

The use of COLTENE Adhesive may only be undertaken by specialist personnel such as dentists and dental assistants with special qualifications.

### Side effects / Interactions

No harmful reactions or secondary effects on patients and / or dental personnel are known.

### Users / patient groups

Application is carried out by dental professionals. Suitable for all patient groups.

### PROPER USE

Line all trays with a layer of Coltène Adhesive as thin as possible and let the applied layer dry for at least 1 min, or dry it with compressed air.

Close bottle immediately after use!

### CLEANING & DISINFECTION

#### Cleaning of trays

The impression material can be removed with a blunt instrument. Soaking in a universal commercial solvent or petroleum ether will dissolve COLTENE Adhesive. These solvents should only be used in a well-ventilated area. The trays can be cleaned and disinfected as usual. Disposable plastic impression trays are not intended to be cleaned.

### Shelf life and storage

COLTENE Adhesive fulfils the intended purpose at least until the expiry date, which is displayed on the containers, in a well-sealed bottle at a temperature of 15-23 °C / 59-73 °F and 50 % relative air humidity. Shelf life after first opening 3 months.

### Disposal

Dispose of waste according to applicable legislation. Do not allow to enter into surface water or drains. Can be burnt together with household waste in compliance with official regulations in contact with approved waste disposal companies and with authorities in charge.

### Reporting obligation

All serious incidents occurring in conjunction with this product must be reported immediately to the manufacturer as well as to the competent authority.

For SDS see  
[www.coltene.com](http://www.coltene.com)



### Glossary



Consult instructions for use



Keep away from sun light



Temperature limitation



Marking of Conformity Europe



Identification for Russia



Identification for Ukraine



Legal Manufacturer



Importer



Expiry Date



RX only



European Authorized Representative



Flame



Environment



Batch Code



Medical Device



Reference Number



Unique Device Identifier

**Coltène/Whaledent AG**   
Feldwiesenstrasse 20  
9450 Altstätten/Switzerland  
T +41 71 757 53 00  
F +41 71 757 53 01  
[info.ch@coltene.com](mailto:info.ch@coltene.com)

 **COLTENE**