

BioSonic® UC150 Ultrasonic Cleaner Performance test

Prepare a fresh tank of cleaning solution according to manufacturer's directions. Run the unit for 5-15 min to degas the solution. The degassing will eliminate any air found in fresh solution. Turn unit off.

Use the pencil load test to check the performance of the device.

Pencil Load Test

1. Rub the surface of an unglazed ceramic disc or plate having a matte finish and a diameter of approximately 2 in (thickness is not critical) with a standard HB pencil lead and then immerse the disc in the ultrasonically agitated cleaning tank. A ground glass stopper, a sheet of ground glass or an aluminum sheet with a thickness of 2 mm to 3 mm may be substituted for the ceramic disc.
2. The ultrasonic cleaner should completely remove the pencil lead within 3 minutes.

Consult your dealer or COLTENE for service if needed.



Coltène/Whaledent AG
Feldwiesenstrasse 20
9450 Altstätten/Switzerland
T +41 71 757 53 00
F +41 71 757 53 01
info.ch@coltene.com
www.coltene.com

US/CA Representative
Coltène/Whaledent Inc.
235 Ascot Parkway
Cuyahoga Falls, OH 44223/USA
T +1 330 916 8800
F +1 330 916 7077
info.us@coltene.com



COLTENE