

HySolate Liquid Dam

Instructions for use

EN

PRODUCT DESCRIPTION

HySolate Liquid Dam is a visible light cure flowable microhybrid composite for protecting the gingiva adjacent to teeth being whitened.

INTENDED USE / CLINICAL BENEFIT

Gingiva protection during bleaching procedures

PERFORMANCE FEATURES

The performance features of the product meet the requirements of the intended use.

CONTRAINDICATIONS

Contraindicated in the case of hypersensitivity to the ingredients of HySolate Liquid Dam.

SAFETY INSTRUCTIONS

NOTICE

- The application needles are single use.
- The ambient light may start polymerization of the material.
- Do not use any resin to adjust viscosity of the material.
- Avoid contact with skin and eyes.
- Unpolymerized material may have an irritant effect and can lead to sensitization against methacrylates.
- Agents containing eugenol may affect the polymerization of HySolate Liquid Dam.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep out of the reach of children!

USER- / PATIENT GROUP

The product shall be used by qualified dental professionals only. Suitable for all patient groups.

PROPER USE

Isolate the gingiva with cotton rolls and gauze. Apply HySolate Liquid Dam directly from the syringe along the gum line using the enclosed application needles. The layer should have a thickness of at least 1-1.5 mm. Light cure with a commercially available light curing unit (wavelength range 400-500 nm, light intensity of at least 1000 mW/cm²) for 10 seconds per arch. Hold the light emission window as close as possible to the material.

After bleaching procedure is finished, remove HySolate Liquid Dam with a spatula or tweezers.

SHELF LIFE / STORAGE

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

DISPOSAL

Dispose of product waste according to applicable legislation.

REPORTING OBLIGATION

All serious incidents occurring in conjunction with this product must be reported immediately to the manufacturer as well as to the competent authority.

WARRANTY





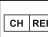





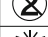


P. L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P. L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P. L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the product.

LIMITATION OF LIABILITY


Except where prohibited by law, P. L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.


SAFETY DATA SHEET

www.coltene.com

Glossary	
	Consult instructions for use
	Marking of Conformity Europe
	Medical Device
	Legal Manufacturer
	Swiss Authorized Representative
	Distributor
	Reference Number
	Expiry Date
	Batch Code
	Unique Device Identifier
	Single use only
	Keep away from sun light
	Temperature limitation



P. L. Superior Dental Materials GmbH 
 Stiller Weg 15 A
 22607 Hamburg, Germany

Coltène/Whaledent AG 
 Feldwiesenstrasse 20
 CH-9450 Altstätten

Customer Center
service@coltene.com