

Adhesive AC

Instructions for use

EN

Please read the Instructions for Use carefully before using the product. Keep for later reference.

PRODUCT DESCRIPTION

COLTENE Adhesive AC is an autoclavable sprayable adhesive for the permanent bond between addition-polymerised silicones and plastic impression trays made of acrylates, polystyrene, polycarbonates.

INTENDED USE

A thin layer of COLTENE Adhesive AC serves as an adhesive between the impression material and the impression tray. It prevents the impression material from detaching from the impression tray during removal.

COMPOSITION

Main component: Polydimethylsiloxanecopolymer

INDICATION

- No direct medical indication.
- Used in combination with medical devices.

CONTRAINDICATION

- In case of known allergies to the main ingredient of COLTENE Adhesive AC, do not use.

SAFETY INSTRUCTIONS

- Use only in well-ventilated areas!
- Keep away from sources of ignitions!
- Always wear gloves.
- The use of COLTENE Adhesive AC may only be undertaken by specialist personnel such as dentists and dental assistants with special qualifications.
- Because of the irreversible bond between addition-polymerized silicones and plastic impression trays, the plastic impression trays sprayed with COLTENE Adhesive AC cannot be reused after the impression is taken.
- COLTENE Adhesive AC is not compatible for use with metal trays or special plastic impression trays, in case of doubt test the acceptability.

SIDE EFFECTS / INTERACTIONS

No harmful reactions or secondary effects on patients and / or dental personnel are known.

USER- / PATIENT GROUP

The product shall be used by qualified dental professionals only. Suitable for all patient groups including children, elderly people and pregnant women.

PROPER USE

Spray (2 - 3 times) all plastic impression trays from a short distance with a thin, even layer of COLTENE Adhesive AC, and let it dry for at least 1 min, or dry it using compressed air.

REPROCESSING, CLEANING, DISINFECTION AND MAINTENANCE

Cleaning of trays

Disposable plastic impression trays are not intended to be cleaned.

SHELF LIFE / STORAGE

COLTENE Adhesive AC fulfils the intended use at least until the expiry date, which is displayed on the containers, in a well-sealed bottle at a temperature of 15 - 23 °C / 59 - 73 °F and 50 % relative air humidity. Shelf life after first opening 3 months.

DISPOSAL

Dispose of waste according to applicable legislation. Do not allow to enter into surface water or drains. Can be burnt together with household waste in compliance with official regulations in contact with approved waste disposal companies and with authorities in charge.

REPORTING OBLIGATION

All serious incidents occurring in conjunction with this product must be reported immediately to the manufacturer as well as to the competent authority.

In the unlikely event of inhalation, ingestion, eye contact, or similar incidents seek immediate medical attention from an appropriate medical specialist to mitigate potential harm.

SAFETY DATA SHEET

www.coltene.com



Glossary	
	Consult instructions for use
	Keep away from sun light
	Temperature limitation
	Marking of Conformity Europe
	Identification for Ukraine
	Legal Manufacturer
	Expiry Date
	Batch Code
	RX only
	Medical Device
	Manufacturing Date
	Unique Device Identifier
	European Authorized Representative
	Importer
	Reference Number

COLTENE International Dental Group

Dent4You AG
Bahnhofstrasse 2
CH-9435 Heerbrugg

Made in Switzerland by
Coltène/Whaledent AG
Feldwiesenstrasse 20
CH-9450 Altstätten

Customer Center
service@coltene.com