# Biosonic<sup>®</sup> UC300

## Ultrasonic Cleaning System

<b>EN</b>	Owner's Guide	3
<b>FR</b>	Guide d'utilisation	7
<b>ES</b>	Guía de uso	11
<b>IT</b>	Guida per l'utente	15
<b>DE</b>	Benutzerhandbuch	19
<b>SV</b>	Bruksanvisning	23
<b>FI</b>	Käyttöohje	27
<b>NL</b>	Gebruiksaanwijzing	31
<b>ZH</b>	所有者指南	35

## An Essential Part of Your Infection Control System.

Congratulations on your purchase of a BioSonic® Ultrasonic Cleaner - the finest, most efficient and most popular ultrasonic cleaning system in dentistry.

Developed specifically for the dental industry, BioSonic® units feature unique product design and optimize time and safety in the office or laboratory.

BioSonic® is a complete ultrasonic cleaning system, with a full line of cleaning solutions and unit accessories, which provides you with the flexibility to efficiently clean instruments, prostheses and other items.

### What is Ultrasonic Cleaning?

Ultrasonic Cleaning is a process created by high frequency soundwaves. The soundwaves, enhanced by specially formulated cleaning solutions, create high-energy cavitation. During cavitation, millions of tiny bubbles form and then collapse or "implode," releasing enormous amounts of energy and shock waves which scour the surface of instruments, appliances and other apparatus placed in the cleaning solution. This powerful scouring action reaches into minute crevices, which manual brushing cannot reach. The combination of energy and specially formulated solutions make ultrasonic cleaning the most effective method for removing gross and microscopic debris.

**Intended Use:** The Biosonic® UC300 Ultrasonic Cleaner is intended for cleaning medical instruments by the emission of high frequency soundwaves.

### I. Unpacking

1. Remove your BioSonic® unit from the shipping container and inspect the unit for any possible damage which may have occurred during shipping.
2. Check the serial number of the unit (found on the underside of the unit) with the number on the shipping carton. If they are not the same contact your dealer.
3. Fill out the warranty card and mail to Coltène/Whaledent within 10 days of receipt.
4. Select your desired location for the cleaner. It should be in close proximity to a sink or waste lines to facilitate ease of draining. The hose supplied with the unit will allow you to place the unit on either side of the sink or connect directly to the waste line.
5. Connect one end of the hose to the drain valve (**Fig. 1**) and place the other end in the sink. If desired, have your plumber connect the hose directly to the waste system with the appropriate fittings.

## II. Operating Instructions

### A. General Cleaning

1. Fill the unit with warm water to the fill line indicator (**Fig. 2**).
  2. Pour the required amount of ultrasonic cleaning solution (either BioSonic® General Purpose, BioSonic® Enzymatic or BioSonic® Germicidal Cleaner) into the tank.
- Note:** Never operate the unit without solution. Operating a dry tank may result in permanent damage and will void the warranty. Do not place objects directly on bottom of tank, use accessory baskets, cassette racks or beaker holder.
3. Plug the line cord into a three-wire grounded AC outlet (or per local regulations).
  4. To activate the unit, turn the timer to the desired cleaning time. The green LED light will glow when the unit is in operation. Unit will automatically shut off at the end of the cycle.
  5. Change the solution in the tank daily, or sooner if it appears soiled or discolored. When beakers are used, change the solution in the beaker after each use.
  6. Drain unit by opening valve (**Fig. 3**) and allow solution to empty into sink or waste system. Rinse tank and close valve. Never drain tank by manually discarding solution as this may cause the unit to stop working properly.

### B. Degassing

Degassing the ultrasonic solution is an important step. Degassing is the removal of large air bubbles found in all fresh solution, which act as a barrier to efficient cleaning.

### Degassing is necessary every time new solution is added.

Degas the solution by activating the unit for 15 minutes before you add any items to be cleaned.

### C. Cover

The BioSonic® Ultrasonic Cleaner should always be covered when operating to trap aerosols in the unit. The BioSonic® UC300 is equipped with a "hinged" cover for easy one hand opening. The cover also conveniently lifts off for cleaning or for use in spaces with low cabinets above the unit.

### D. Instrument Draining

The UC300 is equipped with a bezel surrounding the tank. The bezel drains condensation back into the tank keeping the work area around the unit dry. The bezel also acts as a draining stand for the large basket or cassette racks. Simply position

the corner of the basket or rack on two of the draining tabs at opposite corners of the tank (**Fig. 4, 5**) and allow the solution to drain off instruments.

## III. Cleaning Methods

### A. In Tank

#### Basket Cleaning:

Place parts to be cleaned into the basket. Then place the basket directly into the main tank containing BioSonic® General Purpose, BioSonic® Enzymatic or BioSonic® Germicidal Cleaner. Clean according to instructions on bottle label.

**Note:** Baskets are furnished with adjustable partitions to keep instrument set-ups separated by user or procedure, eliminating sorting and speeding instrument processing. For more information, see basket accessory directions or basket section insert directions.

#### Cassette Cleaning:

Place filled rack (**Fig. 6**) into tank containing BioSonic® General Purpose, BioSonic® Enzymatic Cleaner or BioSonic® Germicidal Cleaner.

### B. In Beaker

#### For specialized item cleaning (e.g. prostheses):

Place item into a beaker, add desired cleaning solution (e.g., BioSonic® Cement Remover) according to the manufacturer's directions. Suspend the beaker in the tank by the use of the beaker holder and place the beaker positioning ring around the beaker so that the bottom 2/3 will be suspended in the main bath.

## IV. Accessory Use Directions

### A. Large FingerGuard Instrument Basket:

BioSonic® UC310 Large FingerGuard Instrument Basket cleans large volumes of instruments and increases instrument processing efficiency.

The Large FingerGuard Instrument Basket comes with five (5) adjustable Basket Section Inserts to help keep instruments sorted by user or procedure, eliminating time consuming instrument sorting. Additional section inserts are available in packets of two (Cat. No. UC335).

#### Use Directions:

The Large FingerGuard Instrument Basket may be used with or without adjustable section inserts. Place instruments across ridges in bottom of basket (**Fig. 7**). Place basket in tank and clean for desired time.

After cleaning, rinse basket and instruments thoroughly under running water. After rinsing, the large FingerGuard basket may be placed on the draining tabs on the bezel for draining (**Fig. 8**).

**Note:** Beaker holder may be used in conjunction with large basket (**Fig. 9**) or twin basket. Remove basket section inserts seated directly under beakers to provide space for beakers to seat fully. Use caution when removing basket if beakers are left in place.

**B. Twin FingerGuard Basket**

The UC311 Twin FingerGuard Basket provides you with the versatility of cleaning two basket loads of instruments or basket and beaker cleaning for prostheses (or other small items). Each Twin FingerGuard Basket comes with two (2) basket section Inserts.

**Use Directions:**

Place Twin FingerGuard Baskets side by side, or use one basket with beaker holder (**Fig. 9,10**). Twin FingerGuard Baskets may be used with or without basket section Inserts. Place instruments across ridges in bottom of basket. Place basket in tank and clean for desired time.

**C. Basket Section Inserts**

The UC335 basket section inserts keep instruments separated by user and procedure, eliminating sorting. This increases efficiency and reduces the potential for accidental sticks and cuts since instruments are pre-sorted and handled less.

**Use Directions:**

Place locking tabs of adjustable inserts into basket holes (**as shown in Fig. 11**). Inserts can be placed at any desired width along the basket. Place as many basket section inserts as required for your cleaning needs. To remove or adjust position, simply squeeze center tabs to unlock section insert from basket holes.

**D. Beaker Holder:**

Place the UC330 beaker holder on bezel. Place beaker positioning rings around the beaker so that at least 2/3 of the beaker is suspended in the main bath (**Fig. 9**). Clean for desired time.

**E. Cassette Racks:**

Your BioSonic® UC300 has two instrument cassette racks to choose from:

**• UC325 Hu-Friedly IMS<sup>1</sup>/ Thompson<sup>2</sup> Cassette Rack**

This rack will hold three (3) large or six (6) small cassettes.

**Use Directions:**

Up to three (3) large cassettes may be placed horizontally in each of the three (3) tiers of the rack (**Fig. 12**). Up to six (6) small cassettes can be placed either horizontally or vertically in the basket (**Fig. 13**). Place filled basket in tank containing BioSonic® General Purpose Cleaner, BioSonic® Enzymatic Cleaner or BioSonic®

Germicidal Cleaner. Once cassettes are cleaned, remove rack from tank and tilt back over tank to drain. Rack has stops on the back to prevent the cassettes from slipping out (**Fig. 14**). Rinse rack and cassettes on all sides thoroughly under running tap water. Place the basket on draining tabs of unit bezel to dry. (See Instrument Draining section for more details.)

**Use Directions:**

Place up to five (5) Steri-Instrument cassettes vertically into rack and place rack into tank for cleaning (**Fig. 15**). For Zirc<sup>3</sup> or Clive Craig<sup>4</sup> instrument boxes, place horizontally in each of the two (2) lower tiers of the rack. Up to ten (10) instruments boxes may be placed in the rack (**Fig. 16**). Place rack in tank for cleaning. After cleaning, rinse cassettes in rack thoroughly under running tap water and place on draining tabs of bezel to dry. (See Instrument Draining section for more details.)

<sup>1</sup> Registered trademark of Hu-Friedly Inc.


<sup>2</sup> Registered trademark of Thompson Inc.

<sup>3</sup> Registered trademark of Zirc Inc.


<sup>4</sup> Registered trademark of Clive Craig Inc.

**V. Environmental Conditions**

- Intended for indoor use
- Maximum altitude 2000m
- Temperature range: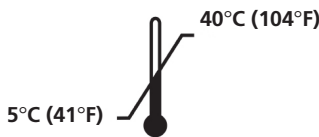

- Humidity Range

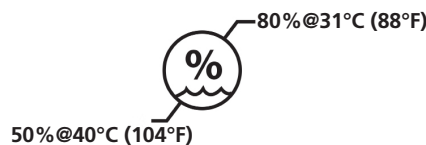- Mains supply voltage fluctuations not to exceed ± 10% of the nominal voltage
- Transient over voltages category II
- Pollution degree 2
- Class I equipment

**VI. ⚠ Cautions and Warning**

**NOTE:** Failure to comply with these Caution and Warning statements may result in injury to personnel or equipment damage.

 <b>CAUTION!</b>	Do not operate the unit without sufficient solution in the tank. Do not use water alone as the carrier bath. Water is not a satisfactory coupling agent unless a wetting agent is used. For best results use BioSonic® General Purpose, BioSonic® Germicidal Cleaner or BioSonic® Enzymatic Ultrasonic Cleaning solutions.
	Do not place objects directly on bottom of the tank because they may block cavitation and cause buildup of heat in the transducer causing premature failure. Use only BioSonic® UC300 accessory baskets, cassette racks or beakers to hold items.
	Do not restrict airflow. Provide adequate ventilation by maintaining at least 2 inches clearance at the back.
	Do not subject unit to shock or impact.

 <b>CAUTION!</b>	Do not immerse the unit in water.
	This product is intended for use only by trained health-care professionals.
	Use this product in accordance with the instructions in the owner's guide.
	Unspecified or improper use of this product may impair safety protection.
	Do not place fingers or hands in the ultrasonic tank while in operation.
	Before cleaning items of considerable value ultrasonically, consult the item's manufacturer for cleaning precautions.
	The power off selection from the keypad does not turn off main power to the unit.
	Never drain tank by manually discarding solution as this may cause the unit to stop working properly.

**CAUTION!**

This equipment may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures such as re-orienting or relocating the unit or shielding the location.

**WARNING!**  
(Failure to comply may result in personnel injury and/or equipment damage.)

Use only the supplied power cord or equivalent

Do not disassemble the unit. There are no user serviceable parts inside.

**WARNING!**  
(Failure to comply may result in personnel injury and/or equipment damage.)

Keep unit covered to reduce airborne contaminants

Do not place fingers or hands in the ultrasonic tank while in operation.

Operation for long periods of time will cause the cleaning solution to get very hot. To avoid scalding, do not place hands in the solution or spill any onto you. Be careful when removing an item from the tank as it may also be hot.

Do not use flammable liquids with Biosonic® units.

**VII. Order Information**

**Unit and accessories**

Cat. No	Description	Contents
UC300	100, Unit (100V, 50/60Hz)	1
	115, Unit (115V, 50/60Hz)	1
	230, Unit (230V, 50/60Hz)	1
	230CE, Unit (230V, 50/60Hz)	1
	240, Unit (240V, 50/60Hz)	1
UC310	Large FingerGuard™ Instrument Basket	1
UC311	Twin FingerGuard Instrument Basket	1
UC325	Cassette Basket for IMS Cassettes	1
UC330	Two Beaker Holder	1
UC335	Basket Section Inserts	2
UC340	Replacement Cover	1
UC350	Replacement Drain Filters	3
UC355	Drain Hose	1
UC53	600 ml Beaker with Cover	1

**Tank Solutions:**

Cat. No	Description	Contents
UC30	General Purpose Solution in MeterDose® Dispensing Bottle	16 oz. (473 ml)
UC31	General Purpose Cleaner + Super Rust Inhibitor	20 Single Dose Packets (15 ml)
UC32	Enzymatic Ultrasonic Cleaner Concentrate in MeterDose Dispensing Bottle	8 oz. (236 ml)
UC34	Plaster & Stone Remover Powder Concentrate in Jar	1.85lbs. (840 g)
UC38	Germicidal Ultrasonic Cleaning Concentrate in MeterDose Dispensing Bottle	8 oz. (236 ml)
UC39	Cement Remover Concentrate in MeterDose Dispensing Bottle	16 oz. (473 ml)
UC42	Germicidal Ultrasonic Cleaning Concentrate in MeterDose Dispensing Bottle	16 oz. (473 ml)

**VIII. Unit Specifications**

Tank capacity:	3.8 gallons (14.4 liters)
Use capacity:	3.0 gallons (11.4 liters)
Overall dimensions:	19" x 12 x 12.25" (48.3 x 30.5 x 31.1 cm)
Tank dimensions:	14" x 9" x 6.75" (35.6 x 22.9 x 17.2 cm)
Power:	300 watts max
Operating frequency:	40 kHz
100-115V Fuse	7A/250V Slow Blow 3 AG
230-240V Fuse	T2.5A/250V, 5x20 mm

**IX. Maintenance**

**A. Cleaning**

All components should be cleaned each day by wiping with a soft cloth moistened with a commercially available mineral deposit remover followed by cleaning with isopropyl alcohol or a mild disinfectant.

**CAUTION!**

Do not use petroleum based solvents, iodophors or phenolic based products. (Iodophors and phenolics can stain the surface of the unit.) Clean up all liquid spills immediately. Rust deposits on tank can be cleaned/removed with a commercially available stainless steel cleaner.

**B. Stainless Steel Cleaning**

**Allowable:**

- Wash with a clean cloth or rag soaked with warm water and mild soap or liquid dish detergent.

- Always clean in the direction of the original polish lines.
- Always rinse well with clear water (2 or 3 times) after cleaning. Wipe dry completely.
- Specialized household stainless steel cleaner may also be used.

**Do Not Use:**

- Steel or stainless steel wool or scrapers to remove stubborn debris.
- Harsh or abrasive cleaners.
- Do not let dirt accumulate.

**Avoid:**

- Cleaners that contain bleach.
- Products that contain chloride, fluoride, iodide, or bromide.

Chemicals harmful to use in the tank:

Acetophenone	Aluminum Chloride
Aluminum Fluoride	Aluminum Sulphate
Aluminum Bifluoride	Ammonium Chloride
Ammonium Hydroxide	Amyl Chloride
Antimony trichloride	Aqua Regia
Bromine	Calcium Bisulfate
Calcium Bisulfite	Calcium Hypochloride
Chloracetic Acid	Chloric Acid
Chlorine Anhydrous	Copper Chloride
Copper Fluoborate	Ethyl Chloride
Ferric Chloride	Ferrous Chloride
Ferris Sulfate	Fluoboric Acid
Flourine	Hydrobromic Acid
Hydrochloric Acid	Hydrocyanic Acid
Hydrofluoric Acid	Hydrofluosilicic Acid
Iodoform	Mercuric Chloride
Muriatic Acid	Phosphoric (crude)
Sodium Hypochlorite	Potassium Chloride
Stannic Chloride	Stannous Chloride
Sulfuric Acid	Zinc Chloride

**C. Inspecting the Power Cord**

Periodically check the power cord for damage and see that the connectors at both ends of the cord are fully seated.

**D. Changing the Fuse**

- Remove the power cord from the wall outlet first and then from the power cord receptacle on the rear of the unit.
- The fuse compartment is located directly next to the power cord receptacle. To access, take a small, flat-headed screwdriver to the tab on the fuse holder. Apply slight pressure to release the fuse holder (**Fig. 17**).
- Gently remove the fuse from the fuse holder.

4. Examine the fuse (**Fig. 18**). If blown, replace with a fuse that matches the unit's voltage and current (see section XIV). If fuse appears to be in good condition return the unit to your dealer or Coltène/Whaledent.
5. Plug the power cord into the power cord receptacle on the rear of the unit. Then plug the power cord into the wall receptacle.

**EMC Class A:** Suitable for use in all establishments other than domestic and those directly connected to a low-voltage power supply network which supplies buildings used for domestic purposes

## X. Symbols

Caution, Hot Surface

Warning/Caution: Refer to accompanying documents

Caution: Risk of electric shock

Proper Disposal Required

## XI. WARRANTY INFORMATION

Our products are carefully manufactured to meet stringent quality assurance requirements. Our products are manufactured from new parts or new and serviceable used parts. Regardless, our warranty terms apply. This product has been developed specifically for use in dentistry and is intended to be operated only by qualified dental professionals in accordance with the instructions contained in this guide. However, notwithstanding anything contained herein to the contrary, the user shall at all times be solely responsible for determining the suitability of the product for the intended purpose and the method of its use. Any guidance on application technology offered by or on behalf of the manufacturer, whether written, verbal or by demonstration, shall not relieve the dental professional of his/her obligation to control the product and to make all professional judgments regarding its use.

Our products are warranted in accordance with the terms of a written Certificate of Limited Warranty accompanying each product. Except for the warranties specifically set forth in the Certificate of Limited Warranty, Coltène/Whaledent Inc. provides no warranties or guarantees of any kind covering the product, expressed or implied, including, without limitation, any warranties as to merchantability or fitness for a particular purpose. The purchaser/user is referred to the Certificate of Limited Warranty for all of the terms, conditions and limitations of the warranty covering this product. This Section of the user manual is not intended to in any way modify or add to the warranty provided in the Certificate of Limited Warranty.

Any claim for damage or breakage to the product in transit should be made to the carrier promptly upon discovery. CW does not warrant the product against shipping damage.

## XII. Disposal

The Biosonic® UC300 is an electric appliance requiring proper disposal according to local laws and regulations.

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© 2015
Coltène/Whaledent GmbH + Co. KG
Coltène AG